

Mucus Extractor – Safety and Performance Information

Warnings:

- Do not insert the catheter past the back of the nose or throat.
- Do not use if packaging is damaged or open, sterility will be compromised, there may be a risk of infection and the device may not perform as intended.
- Do not use if you observe any damage to, blockage of, or foreign bodies in the device as the device may not perform as intended or cause injury to the patient, user or other.
- Do not use if you observe any kinking in the tubing as this may reduce suction and the device may not perform as intended.
- Graduations on specimen bottles are for indication purposes only. The device does not have a measuring function.
- Device is intended for specimen collection only. Once sample is obtained remove the suction catheter/cannula from the patient.
- Device is not intended to preserve samples. Do not use the device to store samples.
- Do not reuse device as there may be a risk of infection to patient, user or other and the device may not perform as intended.
- Device is intended for single patient use.
- Follow local policies and procedures to store and transport samples for further examination.
- Do not use the device past its expiry date. As it may impact the performance of the device.
- If you experience problems using the device, contact your healthcare professional for advice. Serious incidents that occur in relation to the devices intended use, should be reported to the manufacturer and local competent authority.

Storage and Disposal:

- Store in a clean, cool, dry place away from direct sunlight.
- Dispose of as clinical waste in-line with local and Government guidelines.